




# Specification



<b>Model No.</b>	<b>AOT-MD-520</b>
<b>Oxygen Flow</b>	<b>1/32L/min ----1L/min</b>
<b>Ozone Concentration</b>	<b>4-92.5 mg/L adjustable with oxygen</b>
<b>Application</b>	<b>Ozone therapy</b>
<b>Voltage, Power</b>	<b>110-220V AC, 15W</b>
<b>Product Size</b>	<b>240*195* 80 mm</b>
<b>Ozone Module</b>	<b>Corona Discharge Titanium electrode &amp; quartz tube</b>
<b>Housing</b>	<b>White Powder Painting</b>
<b>Package</b>	<b>29.5*32*14cm</b>
<b>Gross Weight</b>	<b>2.5 kg with package</b>
<b>Warranty</b>	<b>1 year</b>

# Diagram:

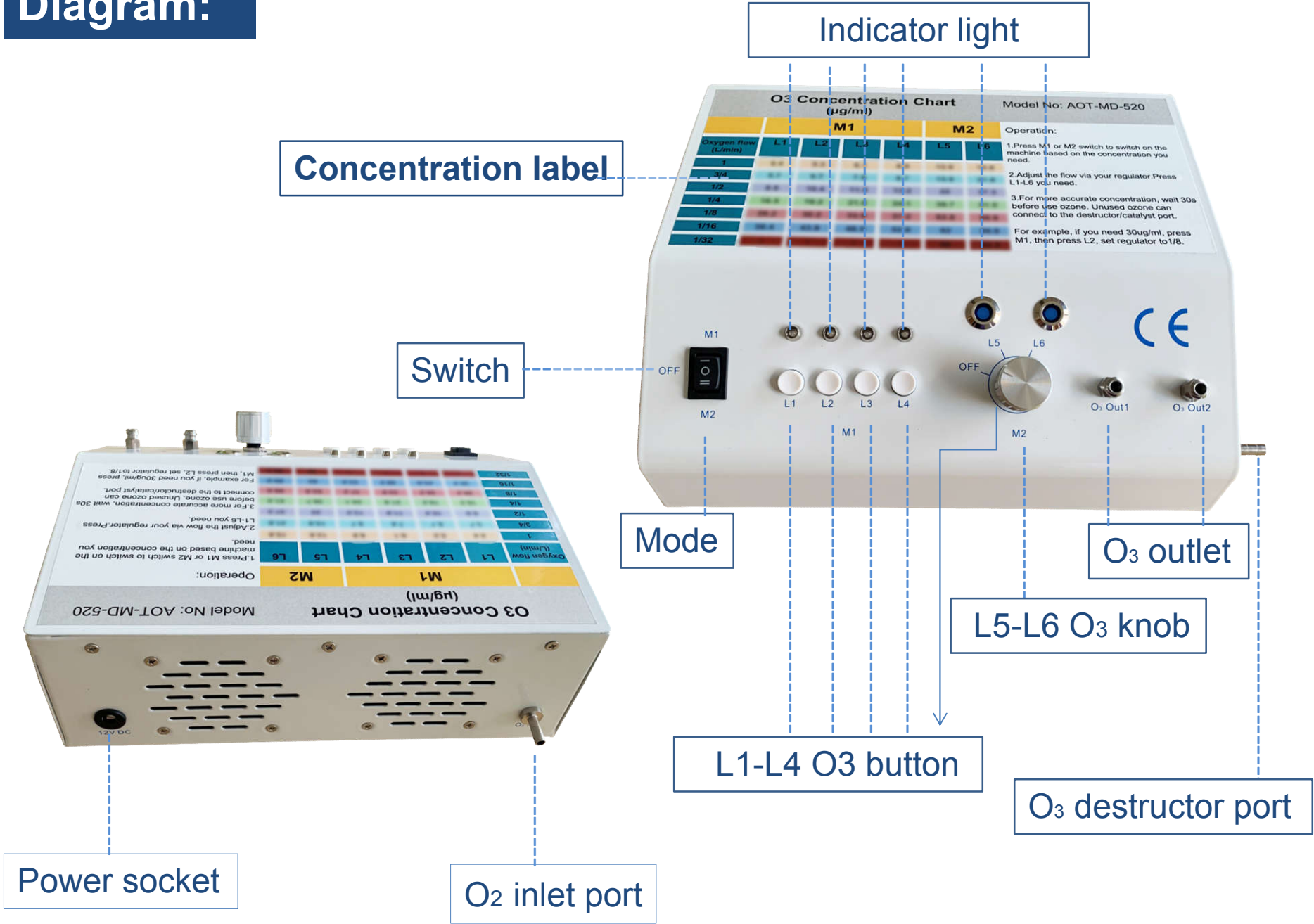

Power socket

O2 inlet port

Concentration label

Switch

Mode

L1-L4 O3 button

L5-L6 O3 knob

O3 destructor port

Indicator light

O3 outlet



## Included accessories:

Male Luer Lock		1 pieces
3-valve lock		1 piece
Silicon hose		2 meters
Stone diffuser		2 pieces
Adaptor 100-240V		1 piece

# Concentration table

Ozone concentration Unit: 1mg/L=1ug/ml=1gamma

## O3 Concentration Chart ( unit : $\mu\text{g/ml}$ )

	M1				M2	
Oxygen flow (L/min)	L1	L2	L3	L4	L5	L6
1	4.4	5.3	6.1	6.9	12.6	15.8
3/4	5.7	6.7	7.8	8.7	15.9	21.6
1/2	8.9	10.4	11.9	13.2	22	37.3
1/4	16.5	19.2	21.8	24.1	39.7	51.5
1/8	26.2	30.2	33.9	37.2	62.8	72.8
1/16	38.4	43.9	48.5	52.0	82	89.9
1/32	/	/	/	/	86	92.5



# Connection view :


(1)



Oxygen cylinder

Flow regulator

O<sub>2</sub>

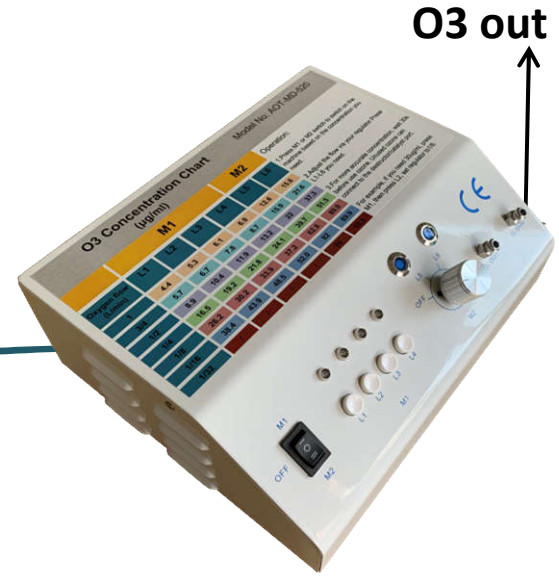

Ozone generator

O<sub>3</sub> out

(2)



Oxygen concentrator


O<sub>2</sub>



O<sub>2</sub> flow meter

With correct flow

O<sub>2</sub>

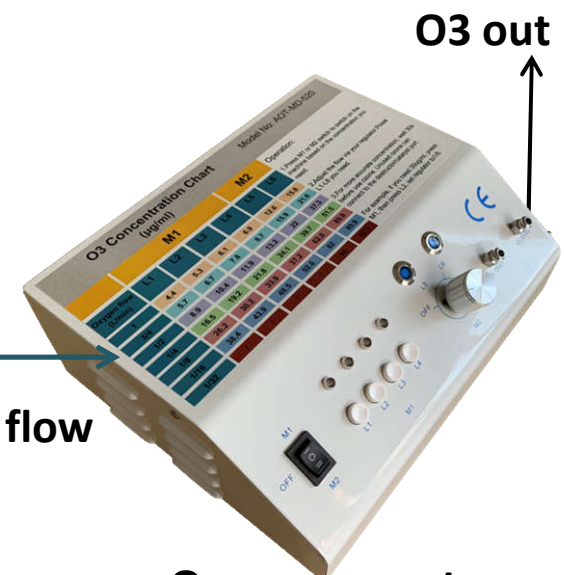

Ozone generator

O<sub>3</sub> out

# Oxygen regulator :



AOT-OR-870



AOT-OR-540



AOT-OR-BN1

AOT-OR-BN2



Oxygen regulator

Oxygen flow: 1/32, 1/16, 1/8, 1/4, 1/2, 3/4, 1, 2, 3, 4 L/min

# Application :



Medical ozone water



Ozone injection



Dental treatment



Major autohaemotherapy



Ear insufflation



Ozone oil



Skin therapy



Saline intravenous injection



Rectal insufflation



Pain therapy




Gynecology



Ozone sauna

# Ozone therapy benefit:

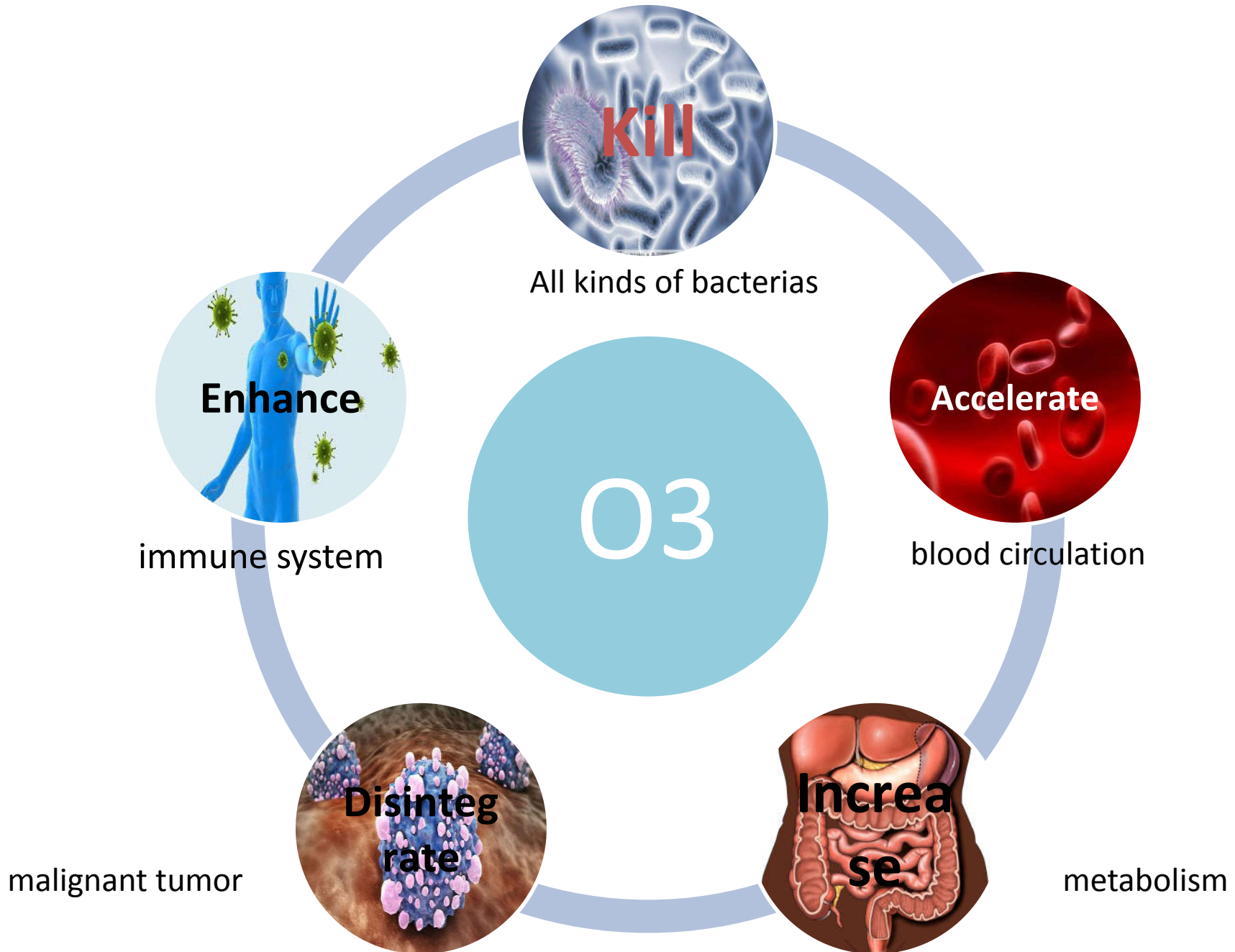

# Certification :

## Certificate of Quality Management System for Medical Devices

Certificate No.30418Q00690ROS

This is to certify that

### Aquapure (Shenzhen) Ozone Technology Co.,Ltd.

**Registration Address:** R1102,University-town Business Park, Lishan Road,Taoyuan, Nanshan District, Shenzhen, China

Which is in conformity with

### YY/T0287-2017 idt ISO13485:2016

#### Scope of Certification

*Technology development, transfer, consultation and service of environmental protection technology products (excluding restricted items), Provide solutions for water purification, sewage treatment and air evolution, and sales of related products, Energy-saving and environmental protection technology development and services, Domestic trade (excluding franchise, monopoly, and special control products),Operating import and export business*

**Issue date:** 18 March, 2021      **Term of validity:** 17 March, 2024

The certification validity period is three years under the condition that the certificate holder's management system consistently meets the quality management system standards.

It is also available at <http://www.zb-iso.com/>. The validity of this certificate shall be maintained by the Center through regular supervision and verification.

Signed by: \_\_\_\_\_

英诺威认证 (中国) 有限公司

<http://www.zb-iso.com>

Certificate – Сертификат – 證明書 – Certificat – 증명서 – شهادة

Shenzhen ZKT Technology Co., Ltd.  
2F, Building 1, Maozhoushan Industrial Park, Houting Community,  
Shajing Street, Bao'an District, Shenzhen, China



## Certificate of Compliance

Certificate Number: ZKT-2019022382C

**Certificate's Holder :** Aquapure (Shenzhen ) Ozone Technology Co.,Ltd  
Room 1102, University-town Business Park, Lishan Road,  
Taoyuan, Nanshan District, Shenzhen, China -518055

**Manufacturer :** Aquapure (Shenzhen ) Ozone Technology Co.,Ltd  
Room 1102, University-town Business Park, Lishan Road,  
Taoyuan, Nanshan District, Shenzhen, China -518055

**Trade Mark :** N/A

**Product :** ozone generator

**Model(s) :** AOT-MD-520

**Test Standard :** EN 55014-1:2017  
EN 61000-3-2:2014, EN 61000-3-3:2013  
EN 55014-2:2015

This Attestation of Compliance is issued on a voluntary basis for electrical equipment below the voltage limits of EMC Directive 2014/30/EU. The essential requirement are fulfilled accordingly based on the technical specifications applicable at the time of issuance. See also notes overleaf. It is only valid in connection with the test report number: ZKT-2019022382E.



This Certificate of Conformity is based on single evaluation of the submitted sample(s) of the above mentioned product. It does not imply an assessment of the whole product and relevant . Directives to be observed.

+86-400 000 9970    +86-755-2233 8688    zkt@zkt-lab.com    www.zkt-lab.com

# Customer Feedback :

Order No.: [82949359001026417](#)

Time Created: 2021-04-29 12:44

Paul Holmes,United States

Supplier Service: **Very Satisfied** (5 stars)

On-time Shipment: **Very Satisfied** (5 stars)



4-95ug/ml Ozone Blood Therapy Machine AOT-...



Great product, very fast shipping, fantastic price. I would recommend this company and this item to anyone looking to acquire a home, medical grade ozone generator. The instruction manual could be better, but if you combine that with the supplied concentration label, and check out... [View More](#)

Order No.: [67159623001021511](#)

Time Created: 2021-01-25 06:37

fabiola eguez,Bolivia

Supplier Service: **Very Satisfied** (5 stars)

On-time Shipment: **Very Satisfied** (5 stars)



ISO13485 Medical Ozone Generator For Blood ...



Excelente cumplimiento de llegada de los productos. gracias.

# Thank you



**AQUAPURE (Shenzhen) Ozone Technology Co., LTD**

**Add:R1102,University Town Business Park, Lishan Road, Taoyuan, Nanshan, 518055 Shenzhen,  
China Office**

**Tel: 0086-755-2660 5963**

**Contact Person: Joey**

**Mob/Wechat/WhatsApp: +86-18844541798**

**Email : Joey.chen@aquapureozone.com**

**Website: www.aquapureozone.com**